


VIDAS® D-DIMER EXCLUSION™ II

CONFIDENTLY RULE-OUT VTE (PE/DVT)* IN SUSPECTED LOW/MEDIUM PTP PATIENTS

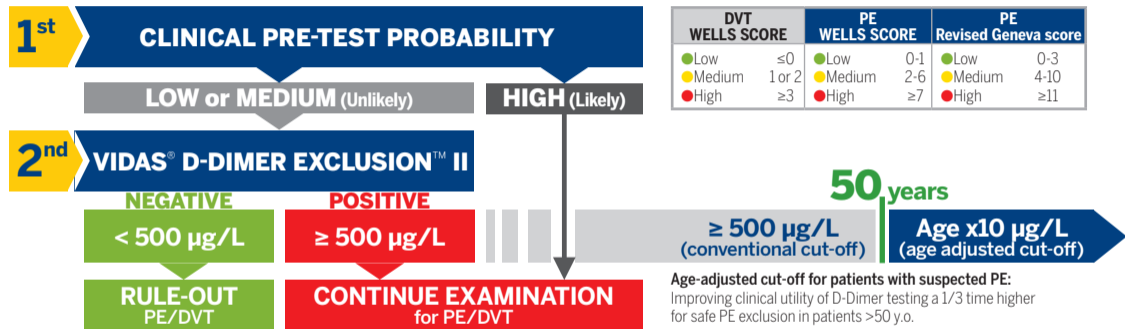

VIDAS® D-DIMER EXCLUSION™ II, with a sensitivity of 100% [95.1-100], a Negative Predictive Value of 100% [95.8-100], combined with PTP, safely excludes up to 50% of patients with suspected PE/DVT in both Low & Intermediate PTP groups

Huisman MV, Klok FA. Thromb Haemost. 2013 • 2019 ESC Guidelines • Douma RA et al. BMJ. 2010 • VIDAS® D-DIMER EXCLUSION™ II package insert: Use of kit must comply with the recommendations in this package insert. A D-Dimer test result does not replace clinical judgment. * Venous Thromboembolism (Pulmonary Embolism/Deep Vein Thrombosis)





VIDAS® D-DIMER EXCLUSION™ II

ANTICOAGULANT TREATMENT GUIDANCE – HERDOO2 RULE

Women **HERDOO2**, Men continue

ASSESS THE RISK OF VTE RECURRENCE IN WOMEN
after completing 5–12 months of **anticoagulant** therapy

HERDOO2 RULE CRITERIA		POINTS
1	HER : any H yperpigmentation, E dema, R edness of either lower extremity	1
2	VIDAS® D-DIMER EXCLUSION™ II : $\geq 250 \mu\text{g/L}$	1
3	O besity: BMI $\geq 30 \text{ kg/m}^2$	1
4	O lder age: ≥ 65 years	1

CAN I SAFELY DISCONTINUE OAT*?

HERDOO2 score = 0 or 1 → Women have a 3% risk of recurrent VTE in the first year after discontinuing OAT.

VIDAS® D-DIMER EXCLUSION™ II package insert • Rodger MA *et al.* BMJ. 2017 • 2019 ESC Guidelines

* Oral Anticoagulant Therapy

0-1 → YES
Discontinue OAT

≥ 2 → NO
Continue OAT

